









# -Scooter Instruction Page-

## Icons on your scooter Dash:

 Indicator "Right"	 "High / Low speed" This limits your maximum speed on whichever setting you are on	 "Speed Knob" Turning towards the turtle is slower Turning towards the rabbit is faster	 "Headlights" Turns headlights on
 Indicator "Left"	 "Hazard Lights" This causes all lights and beeps to go off repetitively until you turn it off	 "Battery Indicator" Green bars or the <b>right side is fully charged</b> . Red or the <b>left side is empty</b> . When it gets to the middle, <b>start to finish your journey</b> .	




Left Hand "pulling towards you" reverses the machine

Right hand "pulling towards you" drives forward


"Pulling towards you" refers to pulling in the direction where you are sitting

Caution: Some driving controls may be inverted / opposite.


**Adjusting the Tiller-** not all machines have the ability to do this, but if the tiller can be adjusted, you will find it in one of three places, as per the arrows indicate.  
a hand brake, knob or hand-wheel tiller adjustment will be in **either**



Hand Brake –Hold this down and move the tiller closer or further away.




Hand-Wheel Tiller Adjustment. Unscrew this, adjust tiller further or closer. And then tighten, being sure to align the "teeth" as you tighten so they do not clash against each other.



Knob–Hold this down and move the tiller closer or further away. This is usually along the "neck" of the tiller.

## Scooter Advice:



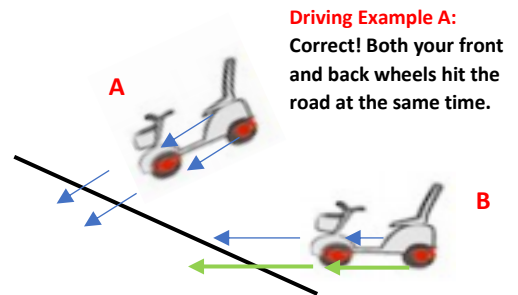

1. Turn your scooter off before getting on/off the scooter
2. Drive slow by checking your **Speed Control Knob** until you are familiar with using your scooter.
3. When crossing roads/footpaths and going over bumps, always cross straight, so both your front wheels touch the road at the same time. This is to **avoid tipping your scooter**. **NEVER Go at an angle**



**Driving Example B:**  
Bad! Your front left wheel hits the road first, tilting your scooter a bit. Then, your right front, and then your back left. Potentially can tip your machine sideways

# -Scooter Instruction Page-

4. When preparing to cross driveways and pedestrian footpaths, do not go too fast, and do NOT go to slow.
  - a. If you stop right before going over the curb, you machine won't have enough power to get you up.
  - b. If you go to fast when going over the curb, your might lose control of your tiller.
  - c. **Always go at a steady pace that can be controlled.**
5. If your scooter can take hills / inclines or you have a steep driveway to get over, **put the speed down when going over these slopes.** The scooter draws a lot of power from the battery when going up these slopes, and to avoid stressing / wrecking the battery and overworking the machine, go at a lower speed. Your machine **will** slow down when going upwards, but this is normal. NEVER go at full throttle up these hills / inclines.
6. You cannot drive the scooter if the charger is in. Please ensure it is out before turning it on.
7. Charge your scooter after each trip. Always be aware of what type of scooter you have. AGM / GEL Deep cycle batteries mean you can leave your scooter on charge continuously. If your machine has **lithium Ion Batteries**, as soon as the charger has turned green, take your scooter off charge immediately.
8. If your machine has lithium Ion batteries, it will be found in the description of each rental available.

 Solax Mobi 1	Portable Manual Folding mobility scooter that folds down easily. It has an adjustable tiller inward and upward and a lightly padded seat for additional comfort. Has a plane safe certificate so ideal for overseas holidays, and compact size makes it perfect for cruise ships! Enjoy the freedom with Solax Mobie <b>Li-Ion battery</b> . 25kg, 10Ah, 8kph, 115kg load.	<a href="#">Make a booking</a> Unavailable	90.00
 i-Moving 3	<b>Suitcase Fold Up Travel Scooter</b> , indoors / outdoors (smooth footpaths, shopping centres, holiday trips).6 Speed, thumb control. <b>Lithium batteries</b> . 12 kph. 150 kg capacity. Splits apart into two easily lifted pieces.	<a href="#">Make a booking</a>	115.00
 Cutie	Small indoor / outdoor portable mobility scooter. Light and suitable for the smaller person, and easily dismantled, totally pull apart and transportable in the smallest of vehicles. Ideal for going shopping. T-bar with light thumb control. 6 kph, 12Ah batteries, 110kg loading.	<a href="#">Make a booking</a>	55.00
 Arika	Fully dismantable, portable, popular. Ideal for forgiving terrain indoors / outdoors (smooth paths, shopping centres, etc). Delta handle bars for those with delicate hands. <b>12 Ah batteries</b> . 8 kph. 130 kg capacity (suitable cruise ships / air travel)	<a href="#">Make a booking</a>	60.00

Batteries	1x 25V 1.06 Ah 274 Wh Li-Ion (Std on all models)
<b>Batteries</b>	<b>2 x 12 V x 18 Ahr</b>

You can see how the top two machines clearly state Lithium ion batteries. Lithium ion Batteries can also be written as "Li-Ion Battery".

You can see how the bottom two just say 12Ah batteries, so they are not lithium.

If you are ever unsure, you can also click on the image or the name of the scooter and have a look in the specifications at the bottom of the new page.

**Alternatively, just ask us or simply take the machine off charge when its fully charged anyway.**

9. **It is VERY important to check, if you are going on a cruise ship, to ensure the volt range is 110V-240V, so your chargers can turn on.**
10. **Do not leave your scooter unattended.** It is your responsibility to pay for a replacement scooter, should it get lost or stolen. Always take the key with you and **ensure you maintain contactable with Mobility Scooters For Hire** for the duration of your hire, should we need to contact you, or visa versa.
11. **If you crash, and parts break off, please pick the pieces up.** We may be able to repair. Your Insurance covers you for 65\$ of damage repair, but NOT replacement of parts you left behind.
12. **It is recommended you download the Scooter Guide for your machine as well, so you have advice beyond the instructions on this sheet and the demonstration our Team gives you, should we be outside business hours. Go to the webpage and select your scooter:**  
<https://www.mobilityscootersforhire.com.au/device-guides.php>
13. The name of your scooter is on the Dashboard or Steering of your scooter, or on your battery box for some portable machines. When speaking to an office attendant about your scooter, please refer to the scooter name or the name of the person the invoice is booked under.
14. **If you require an extension, please contact us 1 day prior to the end date. Your booking dates are on your invoice, so please keep track of your dates.**
  - a. **Please note we will usually charge your credit card for payment of the extension invoice, so please ensure you have funds available on the day of extension if you have asked for one.**