

Scotmobiel Assembly Guide

Assembly of Head Section



There are TWO plugs to pay attention to, and the plugs are colour-coded.




From the headboard:

- One plug is the red and black plug, which **connects into one of the open plugs on the tiller (same colour).**
- The second plug is Red, black and green, and as the previous plug, **connects to the other open plug on the tiller.**

NOTE: Failure to connect to the right colour-coded plugs may result in scooter failure or short, and will not be covered under warranty.

Once you have connected the PLUGS, PLEASE CHECK that the lights and battery indicator turns on by using the set of keys provided with the scooter.

To attach the headboard to the tiller, simply open up the **round clamps** underneath and wrap them around the handlebars. Then, use the included **screws** to tighten the clamps together to hold it.

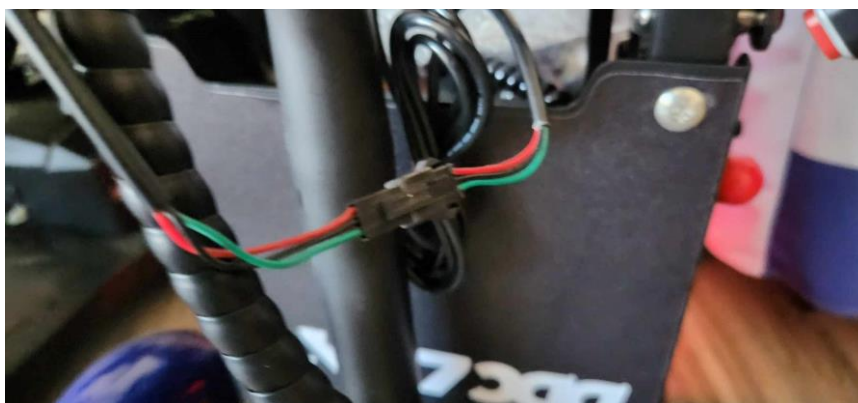

Assembly of the basket



To assemble the basket on the mobility scooter, you have two rounded screws and washers that lock the lid on top of the basket.



Then you must line up the metal piece that holds the basket and secure it using a bolt and nut. That allows the basket to then hook onto the Scootmobiel underneath the seat area.

The images below are to show how it looks.

