Rollator/Transit Wheelchair

equipmed

2 IN 1

SMARTROLLER

equipmed,

equi?Med.



READ THIS MANUAL CAREFULLY BEFORE USE – FAILURE TO DO SO MAY RESULT IN INJURY, PROPERTY DAMAGE AND MAY VOID WARRANTY. • KEEP THIS MANUAL FOR FUTURE REFERENCE. • Products covered by this manual may vary in appearance, assembly, inclusions, specifications, description, and packaging.



Safety

Safety messages are designed to alert you to possible dangers or hazards that could cause death, injury or equipment or property damage if not understood or followed. Safety messages have the following symbols:



You WILL be KILLED or SERIOUSLY INJURED if you do not follow instructions.

It is vital that you read and understand this user manual before using the product, including safety warnings, and any assembly and operating instructions. Keep the manual for future reference.

Safety precautions and recommendations detailed here must be fully understood and followed to reduce the risk of injury, fire, explosion, electrical hazard, and/or property damage.

Safety information presented here is generic in nature – some advice may not be applicable to every product. The term "equipment" refers to the product, be it electrical mains powered, battery powered or combustion engine powered.

- Before Use If you are not familiar with the safe operation/handling of the equipment or are in any way unsure of any aspect of suitability or correct use for your application, you should complete training conducted by a person or organization qualified in safe use and operation of this equipment, including fuel/electrical handling and safety.
- Do NOT operate the equipment in flammable or explosive environments, such as in the presence of flammable liquids, gases or dust. The equipment may create sparks or heat that may ignite flammable substances.
- Keep clear of moving parts.
- Equipment may be a potential source of electric shock or injury if misused.
- Do NOT operate the equipment if it is damaged, malfunctioning or is in an excessively worn state.
- Do NOT allow others to use the equipment unless they have read this manual and are adequately trained.
- Keep packaging away from children risk of suffocation! Operators must use the equipment correctly. When using the equipment, consider conditions and pay due care to persons and property. General Work Area Safety

Concra Work Area Safety

- Work areas should be clean and well it.
- Do not operate the equipment if bystanders, animals etc are within operating range of the equipment or the general work area.
- If devices are provided for connecting dust extraction / collection facilities, ensure these are connected and used properly. Dust collection can reduce dust-related hazards.



You CAN be KILLED or SERIOUSLY INJURED if you do not follow instructions.

General Personal Safety

- Wear appropriate protective equipment when operating, servicing, or when in the operating area of the equipment to help protect from eye and ear injury, poisoning, burns, cutting and crush injuries. Protective equipment such as safety goggles, respirators, non-slip safety footwear, hard hat, hearing protection etc should be used for appropriate equipment / conditions. Other people nearby should also wear appropriate personal protective equipment. Do not wear loose clothing or jewellery, which can be caught in moving parts. Keep hair and clothing away from the equipment.
- Stay alert and use common sense when operating the equipment. Do not overreach. Always maintain secure footing and balance.
- Do not use the equipment if tired or under the influence of drugs, alcohol or medication.
- This equipment is not intended for use by persons with reduced physical, sensory or mental capabilities.

General Fuel Safety

- Petrol/fuel/gasoline is extremely flammable – keep clear of naked flames or other ignition sources.
- Do not spill fuel. If you spill fuel, wipe it off the equipment immediately – if fuel gets on your clothing, change clothing.
- Do NOT smoke near fuel or when refuelling.
- Always shut off the engine before refuelling.
- Do NOT refuel a hot engine.
- Open the fuel cap carefully to allow any pressure build-up in the tank to release slowly.
- Always refuel in well ventilated areas.
- Always check for fuel leakage. If fuel leakage is found, do not start or run the engine until all leaks are fixed.

General Carbon-Monoxide Safety

- Using a combustion engine indoors CAN KILL IN MINUTES. Engine exhaust contains carbon-monoxide a poison you cannot smell or see.
- Use combustion engines OUTSIDE only, and far away from windows, doors and vents.



You CAN be INJURED if you do not follow instructions or equipment damage may occur.

General Equipment Use and Care

- The equipment is designed for domestic use only.
- Handle the equipment safely and carefully.
- Before use, inspect the equipment for misalignment or binding of moving parts, loose components, damage or any other condition that may affect its operation. If damaged, have the equipment repaired by an authorised service centre or technician before use.
- Prevent unintentional starting of the equipment - ensure equipment and power switches are in the OFF position before connecting or moving equipment. Do not carry equipment with hands or fingers touching any controls. Remove any tools or other items that are not a part of the equipment from it before starting or switching on.
- Do not force the equipment. Use the correct equipment for your application.
 Equipment will perform better and be safer when used within its design and usage parameters.
- Use the equipment and accessories etc. in accordance with these instructions, considering working conditions and the work to be performed. Using the equipment for operations different from those intended could result in hazardous situations.
- Always keep equipment components (engines, hoses, handles, controls, frames, housings, guards etc) and accessories (cutting tools, nozzles, bits etc) properly maintained. Keep the equipment clean and, where applicable, properly lubricated.
- Store the equipment out of reach of children or untrained persons. To avoid burns or fire hazards, let the equipment cool completely before transporting or storing. Never place or store the equipment near flammable materials, combustible gases or liquids etc.
- The equipment is not weather-proof, and should not be stored in direct sunlight, at high ambient temperatures or locations that are damp or humid.
- Do not clean equipment with solvents, flammable liquids or harsh abrasives.
- For specific equipment safety use and care, see Equipment Safety.

General Electrical Safety

- Inspect electrical equipment, extension cords, power bars, and electrical fittings for damage or wear before each use. Repair or replace damaged equipment immediately.
- Ensure all power sources conform to equipment voltage requirements and are disconnected before connecting or disconnecting equipment.
- When wiring electrically powered equipment, follow all electrical and safety codes.
- Wherever possible, use a residual current device (RCD).
- High voltage / high current power lines may be present. Use extreme caution to avoid contact or interference with power lines. Electrical shock can be fatal.

General Electrical Safety

- Electrically grounded equipment must have an approved cord and plug and be connected to a grounded electrical outlet.
- Do NOT bypass the ON/OFF switch and operate equipment by connecting and disconnecting the electrical cord.
- Do NOT use equipment that has exposed wiring, damaged switches, covers or guards.
- Do NOT use electrical equipment in wet conditions or in damp locations.
- Do NOT use electrical cords to lift, move or carry equipment.
- Do NOT coil or knot electrical cords, and ensure electrical cords are not trip hazards.

General Service Information

equipmet

- The equipment must be serviced or repaired at authorised service centres by qualified personnel only.
- Replacement parts must be original equipment manufacturer (OEM) to ensure equipment safety is maintained.
- Do NOT attempt any maintenance or repair work not described in this manual.
- After use, the equipment and components may still be hot – allow the equipment to cool and disconnect spark plugs and/or electrical power sources and/or batteries from it before adjusting, changing accessories or performing repair or maintenance.
- Do NOT adjust while the equipment is running.
- Perform service related activities in suitable conditions, such as a workshop.
- Replace worn, damaged or missing warning/safety labels immediately.


Wheelchair Safety

- It is important to keep yourself safe at all times when using your rollator especially when walking on slopes, inclines or while negotiating steps or kerbs.
- Please select your route carefully to avoid uneven or soft ground wherever possible. Don't use the rollator on an incline if covered in snow, ice or a film of water or oil.
- You must not make any unauthorised adaptations or changes to your rollator as this may cause it to become unstable. Practice on level ground as well as going up and down hills slowly.
- Only one person can use the rollator in normal use. Never stand on the frame, seat or wheels when in use.
- Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents. Users may put themselves at risk by improperly
 using their rollator.
- The handbrakes should be used for slowing the rollator down when in use. Always remember to apply the parking brakes when stationary, except when storing the rollator for long periods of time as this could potentially damage or stretch the brake cables.
- In the event of any damage or needed repair, immediately do not use your rollator for any purpose.
- · Always make sure you fully assemble and check your rollator carefully before use.
- · Do not stand directly on the wheels, do not place any weight directly on the wheels.
- MAXIMUM WEIGHT CAPACITY: 120KG



Safety Symbols

The product may have safety warning labels attached to it, explained below. Understand the symbols on your product and their meanings. If any stickers become unreadable, unattached etc., replace them.

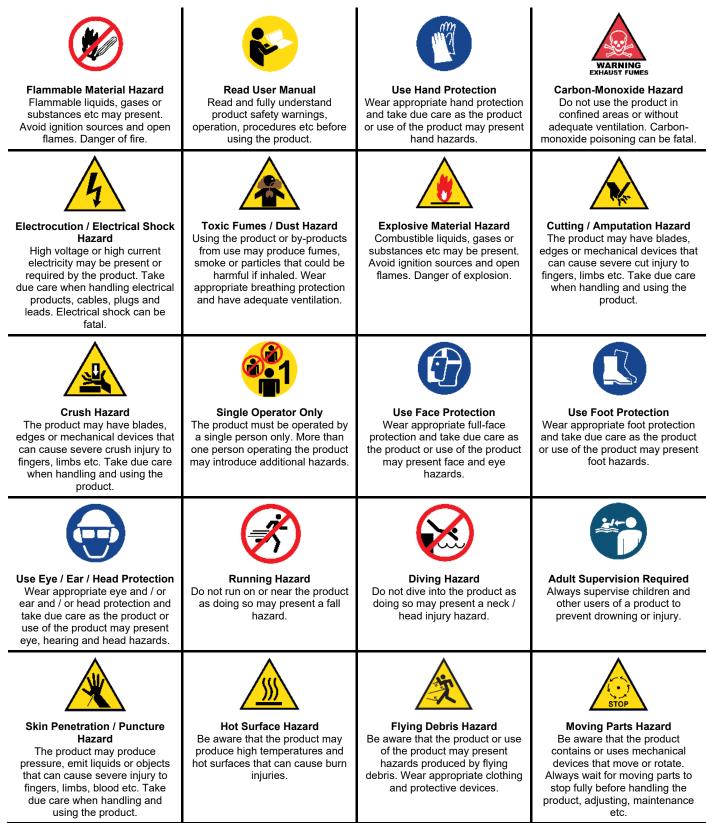

equipMed.

| Carbon-Monoxide Hazard Do not use the product in confined areas or without adequate ventilation. Carbon- monoxide poisoning can be fatal. | Pull Hazard Be aware that the product contains or uses mechanical devices that can pull in objects and can cause severe injury to fingers, limbs etc. Take due care when handling and using the product. | Slope / Fall Injury Hazard Be aware that using the product on sloping surfaces or in slippery conditions may present additional dangers from falls and contact with blades, moving parts, hot surfaces etc. | "Slam Dunk" Warning Do NOT attempt "slam dunk" manoeuvres as this may result in severe injury due to falling, product breakage or collapse etc. |
|---|--|---|--|
| Electrocution / Electrical Shock Hazard - Outdoor High voltage or high current electricity may be present or required by the product. Do NOT use in rain, damp or wet conditions. Electrical shock can be fatal. | Electrocution / Electrical Shock Hazard - Disconnect High voltage or high current electricity may be present or required by the product. Always disconnect the product from the electrical supply before handling the product, adjusting, maintenance etc. | Power Line Electrocution Hazard High voltage / high current power lines may be present. Use extreme caution to avoid contact or interference with power lines. Electrical shock can be fatal. | Kick-Back" Hazard High level of "kick-back" hazard that can cause the machine to suddenly rotate towards operator. Kick-back injury can be fatal. |
| Winch Operator Position Hazard Do NOT stand between winch and load. Do NOT use winch to move people. | Winch Lift Hazard Do NOT LIFT load vertically. Use machine to PULL only. | Cable Hazard Ensure that load bearing cable is not kinked or knotted. | Winch Cable Hazard Ensure that there is a minimum number of cable coils on winching mechanism. |
| Winch Hook Hazard Carry hook to load – do NOT throw or run. | Flash / Blinding Hazard Wear appropriate eye protection for welding. Direct exposure to weld arcs may cause permanent eye injury. | Laser Hazard Laser may be in use – do NOT look directly at laser or allow others to. | |



Table of Contents

| Safety | 2 |
|-----------------------------------|----|
| Safety Symbols | |
| Parts Identification and Features | 7 |
| Spare Parts | |
| Assembly | 9 |
| Maintenance | 11 |
| Specifications | 11 |

EQUIPMED

EQUIPMED EQUIPMED

FOURMED FOURMED

Parts Identification and Features

Lightweight aluminium frame



| 1 | Ergonomic handgrips | |
|---|--------------------------------|--|
| | | |
| 2 | Lockable hand brakes | |
| 3 | Cane holder | |
| 4 | Solid, puncture-proof tyres | |
| 5 | Height-adjustable footplates | |
| 6 | Foldable & removable footrests | |
| 7 | Carry bag | |
| 8 | Deluxe padded seat | |
| 9 | Adjustable back support | |
| | | |
| | | |



8





Spare parts available for purchase, scan QR codes below:



QR Code



SKU

AGCROLEQMAWH1





AGCROLEQMAWH2





AGCROLEQMAWH3

© 2022 Equipmed

2 in 1 Rollator/Transit Wheelchair - SmartRoller

equipMed.

Assembly

1. Open the main frame so the seat has locked into position, as per the image below. Then, place the main frame of the rollator on a solid level surface.



2. Insert the two height adjustable handle sections into the main frame marked "A" in the image below. Make sure the snap button has locked into position at your preferred height setting.



3. Insert the two rear brake wheels into the main frame marked "B" in the image above. Make sure the snap button has locked into position securely.

2 in 1 Rollator/Transit Wheelchair - SmartRoller

equipMed.

4. Insert the two front wheels into the main frame marked "C" and then turn the star knob clockwise, as shown in the image below, until it locks into position. Make sure the screws are fully secured and not loose in anyway.



5. Attach the footrests provided onto the front of the frame and the assembly is complete.



IMPORTANT! Ensure all wheel axles are tight and secure. Do not stand directly on the wheels, do not place any weight directly on the wheels.

2 in 1 Rollator/Transit Wheelchair - SmartRoller

equipmed.

Maintenance

IMPORTANT! Ensure the wheel axles are checked and tightened regularly.

- Do regular checks to ensure that bolts, nuts, and other components are intact and tightened.
- The tyres should have enough air in them to maintain air pressure.
- Use a damp cloth to wipe away any dirt on the wheelchair. Dry with a dry, clean cloth to prevent rusting.

Specifications

| Foldable | Foldable backrest |
|----------------------------------|--|
| Frame material | Aluminium |
| Frame coating | Liquid painting |
| Size of wheels (front) | 8 inches (20cm) |
| Type of wheels (front) | PVC |
| Size of wheels (rear) | 8 inches (20cm) |
| Type of wheels (rear) | PVC |
| Material front fork material | PA + 30% GF |
| Material and type footplate | PA + FIBRE |
| Type of Brake | Hand brake |
| Numbers of brake | 2 |
| Cross | None |
| Armrest | No |
| Leg rest | Swing-away removable footrests and height-adjustable/foldable footplates |
| Seat and backrest | Backrest size: 690x145mm; Seat size: 460x310mm |
| Cushion | No |
| Weight capacity (kg) | 136kg |
| Quick release wheels | No |
| Locking park brakes | Yes |
| Adjustable height of front wheel | No |
| Seat belt | No |
| Seat width | 46cm |
| Carry bag | Yes |
| Umbrella holder/Cane holder | Yes |
| Angle adjustable back support | Yes, back and forward |
| Handle height adjustable | Yes, 85-98cm |
| | |



Some experts believe that the incorrect or prolonged use of almost any product may cause serious injury or death. To help reduce your risk of serious injury or death, refer to the information below. For more information, see www.datastreamserver.com/safety

- Consult all documentation, packaging and product labelling before use. Note that some products feature documentation available online. It is recommended to print and retain the documentation.
- Before each use, check the product for loose/broken/damaged/missing parts, wear or leaks (if applicable). Never use a product with loose/broken/damaged/missing parts, wear or leaks.
- Products must be inspected and serviced (if applicable) by a qualified technician every 6 months. This is based on average residential use by persons of average size and strength, and on a property of average metropolitan size. Use beyond these recommendations may require more frequent inspections/servicing.
- Ensure that all users of the product have completed a suitable industry recognised training course before being allowed access to the product.

- The product has been supplied by a general merchandise retailer that may not be familiar with your specific application or description of application. Be sure to attain third-party approval from a qualified specialist for your application before use, regardless of any assurances from the retailer or its representatives.
- This product is not intended for use where fail-safe operation is required. As with any product (for example, automobile, computer, toaster), there is the possibility of technical issues that may require the repair or replacement of parts, or the product itself. If the possibility of such failure and the associated time it may take to rectify could in any way inconvenience the user, business or employee, or financially affect the user, business or employee, then the product is not suitable for your requirements. This product is not intended for use where incorrect operation or a failure of any kind, including but not limited to, a condition requiring product return, replacement, parts replacement or service by a technician may cause financial loss, loss of employee time or an inconvenience requiring compensation.
- If this product has been purchased in error when considering the information presented here, contact the retailer directly for details of their returns policy, if required.



©2022 Equipmed. All rights reserved. No part of this document, including descriptive content, concepts, ideas, diagrams or images may be reproduced or transmitted in any form or by any means, electronic or mechanical, including photocopying, scanning or recording, or any information storage and retrieval system, without express permission or consent from the publisher.