

## care and maintenance

- Ensure the rollator is used only as a mobility aid.
- Ensure the rollator is kept clean. Wipe down the rollator with a damp cloth on a regular basis.
- Check the moving parts (e.g. wheels and height adjustment) regularly
- All fasteners (screws, nuts and bolt) are to be checked regularly to ensure they are securely tightened.
- Check that the front castor wheels swivel freely. Withdraw the rollator from use if they do not
- Adjusting the footrests length (745 model only):

When adjusting the length of footrests the user should be sitting in rollator facing forward with feet on the footplates. Push the pins on the footrest to lengthen. Raise or lower foot plate to desired position that will provide maximum support of thighs. The footplates should not be so low that they causes uncomfortable pressure to the back of the thighs. Footplates should not be lower than 60mm from the ground to ensure clearance.

## specification

- handle height (max) 37.5" / 95cm
- handle height (min) 31.5" / 80cm
- width 24" / 61cm (seat 18" / 45cm)
- depth 25.5" / 65cm (seat 9" / 23cm)
- seat height 21" / 53cm
- weight limit 300lb / 136kg
- 

*The above dimensions are without footrests*

## warranty details

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from date of purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the customer.

In the event of a defect covered by this warranty, we will at our option supply parts or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, castors and grips, which are subject to normal wear and need periodic replacement.

If you have any queries or require full warranty conditions, please contact your Drive stockist.



### Drive Medical Ltd

Ainley's Industrial Estate, Elland,  
West Yorkshire, GB Hx5 9JP  
[www.drivemedical.co.uk](http://www.drivemedical.co.uk)

## diamond rollator

A large, vertical version of the Drive Medical logo, with the word "drive" in white lowercase letters on a dark blue background. A small yellow and orange graphic element is positioned above the letter 'i'.



[www.drivemedical.co.uk](http://www.drivemedical.co.uk)


## assembly instructions

1. First remove all contents from the carton: backrest, handle with tube, rollator frame and basket, footrests (745 only)
2. Open frame by pushing down on sling seat on both sides until the rollator is open all the way.
3. Turn the rollator upside down so both handgrips are on the floor, and the front and rear castor receptacles are facing upwards.
4. Insert the rear castor by inserting "A" into "B" with the castors turned upwards (figure 1). The rear castor will be positioned properly when the brass release buttons "click" through the frame.
5. Insert lock release knob (figure 2) to lock castor assembly into place.
6. Repeat steps 4 and 5 to install the other rear castor.
7. To install front castors, insert castor leg into front castor housing until a "snap" is heard (figure 3).
8. To adjust handle height, depress brass lock pin on frame (figure 4) and either pull or push handle to desired height. Be sure to repeat process for other handle and ensure both lock pins are in the same position.

### Installing the footrests (745 model only)

Align the two holes of the footrest on top of the frame pins and rotate forward until you hear the lock of the release lever.

To remove the footrest, push the release lever and swing the footrests to the side and lift the footrests off the frame pins.

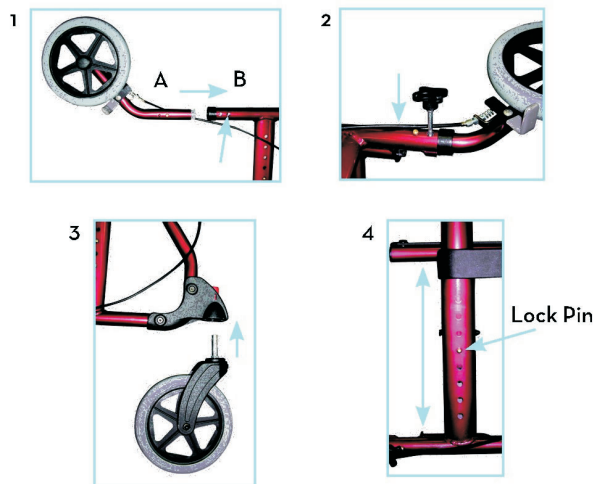

## safety procedures

1. Make sure the rollator is fully opened, the seat is down and the folding bar underneath the seat is locked in the down position.
2. Ensure that both knobs are fully tightened and hand grips are at equal heights. Test the rollator for stability before applying full weight to hand grips.
3. Test the brakes lock before operating the rollator. When the wheels are locked (see illustrations for locking and releasing brakes right), lock the bar on rear wheels to prevent the wheels from turning. If the wheels turn while the brakes are locked, return the unit to your dealer for adjustment. When the brakes are released, the wheels should turn freely.
4. Ensure that good health and safety practice is used for carrying and lifting items, such as the rollator.

5. Always lock the brakes before sitting and do not use seat when the unit is on a gradient or uneven ground.
6. Replace the tyres when significantly worn or damaged.
7. Only the 745 rollator (with footrests) should be used as a transport chair. The device should only be used as a transport chair on flat ground, and the user should sit on the seat and refrain from putting excessive weight on the footrests. Do not have anyone push you while you are sitting back wards on the rollator. You can safely be pushed only when you are sitting forward and you feet are correctly positioned in the footrests.
8. For your health and comfort, make sure that you adopt a comfortable posture and that you can operate the braking mechanism with ease.

## brake operation

Always engage and disengage the brakes on both sides simultaneously:

1. To slow down or come to a temporary stop. Pull up slowly on the brakes with four fingers until the rollator slows down or stops.
2. To apply the parking brakes, press down on both brake handles with your palms until you hear a "click".
3. To release the parking brakes. Push up on both brake levers with the palms of your hands until they are in the original position.

