

Mobility Scooters

for hire sale service

Melbourne



Ronica Trade Pty Ltd®
Factory 25 / 58 Mahoneys Rd
Thomastown, VIC 3074
Australia

Phone: (03) 7036 4440
<http://mobilityscootersforhire.com.au> ®
admin@mobilityscootersforhire.com.au
ABN: 26 939 202 303
ACN: 628 942 727

Dear customer,

Please be aware you are using a high voltage and high power charging device to charge your mobility scooter.

As such, adhere to the following instructions:

1. ALWAYS put your charger into your scooter first.
2. Put the other end of the charger into your wall socket.
3. Turn the power switch on at the wall.

To remove the charger, follow the instructions in reverse order (3, 2, 1).

Failure to do so may result in harm to your device, or to yourself or surroundings.

FINAL NOTE:

DO NOT LET YOUR BATTERY GO FLAT OR TOO LOW. Pay attention to the VOLTAGE display, NOT to the battery indicator.

Please FOLLOW the FAQ on the product's website page for VOLTAGE information. FAILURE to follow these instructions, or letting your battery get too low / empty may result in harm or failure to your device, charger, yourself or surroundings.