

142x210mm



Electric Scooter

User Manual

PARTS LIST



- Max Support: 330 LBS (150 KG)**
- | | |
|----------------------|-----------------------------|
| ① Rearview Mirror | ⑥ Seat Adjusting Handle |
| ② Key | ⑦ Backrest Adjusting Handle |
| ③ Brake Lever | ⑧ Rear Box Support Bracket |
| ④ Charging Inlet | ⑨ Rear Box |
| ⑤ Anti-tipping Wheel | ⑩ Armrest |
| | ⑪ Headrest |

SCOPE OF APPLICATION

The electric scooter is suitable for individuals with limited mobility either indoors or outdoors. It can help the elderly travel more easily and conveniently.

INSTALLATION

This electric scooter comes with the main part and the accessories.



Main Part

Accessories:

Rearview Mirror, Adapter, Headrest, Rear Box, Key, Wrench, L-Wrench, Screws and Nuts, User Manual



1. Install the Headrest

Press down the lock buttons on top of the backrest to insert the headrest, release the buttons to lock the headrest in a desired height. (as shown)



2. Install the Rear Box

Open the rear box and fix it onto the support bracket with 5 sets of screws and nuts, and tighten them with the wrench and L-wrench. (as shown)



3. Install the Rearview Mirrors

Screw the rearview mirrors clockwise into the holes on the handles, and adjust them to a proper position. (as shown)



4. Turn On the Power

Insert the key into the rear keyhole, turn clockwise, then lift the seat and flip the switch to ON (short line). You will hear a beeping sound to indicate the scooter power is connected. (as shown)



OPERATION

1. Insert the key into the front keyhole, turn clockwise to turn on the scooter. Detailed instructions are as shown in the picture below.



- | | |
|----------------------------|------------------------------|
| ① Forward/Backward Switch | ⑥ Display Panel |
| ② Front Light Switch | ⑦ High/Low Beam Control Gear |
| ③ Speed Control Handle Bar | ⑧ Rear Wheel Brake Lever |
| ④ Front Wheel Brake Lever | ⑨ Signal Light Switch |
| ⑤ Speed Control Gear | ⑩ Horn |
| | ⑪ USB Charging Port |

2. Parking / Unparking

Press and hold the rear wheel brake lever, press down the parking button to park the scooter; press the rear wheel brake lever again to let the parking button pop out and unpark the scooter. (as shown)



3. Seat Adjusting

Pull up the seat adjusting handle to move the seat back or forth as desired, release the handle to lock its position. (as shown)



4. Backrest Adjusting

Pull up the backrest adjusting handle to adjust the backrest angle, release the handle to lock its position. (as shown)



5. Armrest Adjusting

Each armrest can be pulled up, adjust the knob underneath to change its angle. (as shown)



PREVENTION AND WARNING

- Product should not be left outside in rainy conditions.
- Please ensure all removable / adjustable parts are in good condition before use.
- Do not use the scooter on solid even surfaces to prevent tipping.
- Necessary aid is needed for users with reduced mobility.
- Do not leave the scooter charging for a long time. An average recharge takes about 8 hours.
- We recommend recharging the scooter before each ride, and when it's not used for an extended period of time, please recharge it once every month.
- Product should be stored in a cool and dry place away from prolonged sunlight.
- Do not use the scooter on driveways or highways.

CLEANING / MAINTENANCE

Do not use strong or corrosive cleaners to clean the product's surface. Clean the scooter exterior with soft, damp cloth and leave it in a well-ventilated area to dry naturally.

LIFETIME LIMITED WARRANTY

The product's main frame is warranted to be free of defects in terms of material and craft during the entire service lifetime.

The device was built to strict standards and carefully inspected prior to shipment. This lifetime limited warranty is an expression of our confidence in the materials and craftsmanship of our products and our assurance to the consumers of years of dependable service.

Your satisfaction is our top priority

Thank you for your purchase – we appreciate having you as a customer! We are committed to providing innovative and affordable mobility solutions to get you moving as soon as possible during daily or after injury uses. We take great pride in helping you!

We hope that you are completely satisfied with your purchase. If you have any issues, please contact us and we will do everything possible to help you. Reviews have a significant impact on our small business and if you purchased this product from an online retailer, we would appreciate a review once you've had a chance to use your new Electric Scooter. We hope we've earned a 4 or 5 star review – if not, please give us the opportunity to help you before leaving a review. Thank you for your support. We appreciate it very much!